

G3 A20 / G3 C20 Quick Operation Manual


1. Button Functions



Home Button: Return to the previous menu or main interface




Knob: Start treatment and adjust device settings

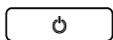



Start / Stop Button: Start / Stop delivering air

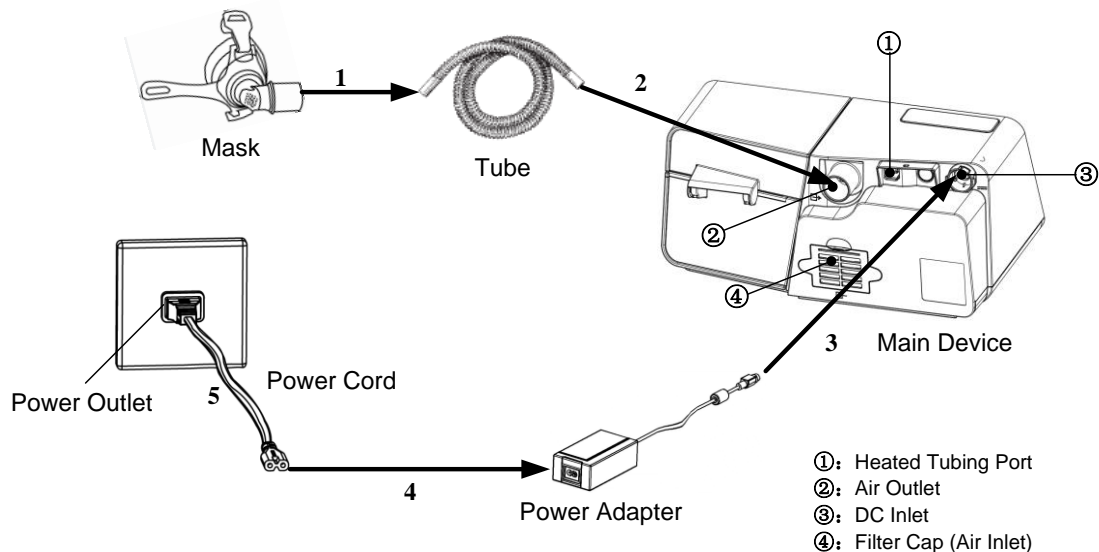
2. Setting up the Device




- Connect one end of the tube to the assembled mask.
- Connect one end of the tube to the air outlet of the device, as shown in the figure above. Connect the heated tube joint to the air outlet of the device, and then insert the power plug into the heated tubing port on the back of the device.
- Connect the power adapter to the DC inlet of the main device.
- Connect the power cord to the power adapter.
- Plug the power cord into the power outlet.

3. Adjusting Device Settings

- When the device is powered on, the display screen will display the Main Interface, as shown in the left figure below. If the heated tube is connected correctly, the icon  will become a number  in the Main Interface on the screen of the device, as shown in the right figure below.

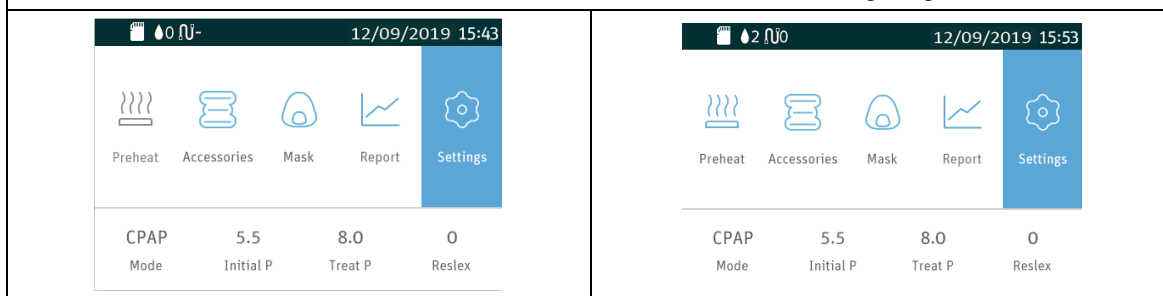

B. Press **the Knob**  to access the Initial Setup Interface of the Patient Menu, as shown in the figure below.



C. Press **the Knob**  to access the Setup Interface, as shown in the figure below.






D. Turn **the Knob**  to adjust an option.




E. Press **the Knob**  to confirm your adjustment to the option.



F. The cursor switches among the options as you turn **the Knob** , as shown in the figure below. Refer to steps C-E to adjust other options. Press **the Home Button**  to exit the Setup Interface.

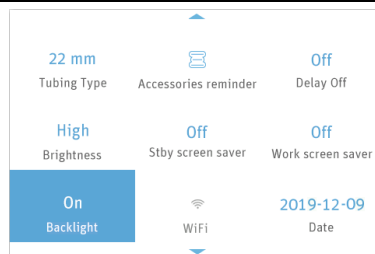

4. Starting Treatment

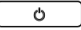After you finish adjusting device settings and return to the Main Interface, press **the Start / Stop Button**  to start treatment.